

ICC 200EA™ Gastrointestinal Settings Chart

Common settings are shown on Units

Polypectomy –
Forced Coag
Snare Resection



- Set AUTOCOAG to Forced
- Set Power: 15-30W
- Use **Blue** Footswitch for bleeders
- Patient plate on patient

OPTION

Polypectomy – Forced
Coag precondition with
ENDOCUT resection



- Set AUTOCOAG to Forced
- Set Power: 25-30W
- Set AUTOCUT to 200W
- Effect 3, ENDOCUT **ON**
- Precondition with **Blue** Footswitch – Tissue will blanch
- Use **Yellow** Footswitch
- Hold pedal down for ENDOCUT resection
- Patient plate on patient


Hot Biopsy with
Forced Coag



- Set AUTOCOAG to Forced
- Set Power: 15-30W
- Biopsy accessory in Mono port
- Tap **Blue** Footswitch
- Patient plate on patient


OPTION

Hot Biopsy with
Soft Coag
Reduces Sticking Preserves Pathology

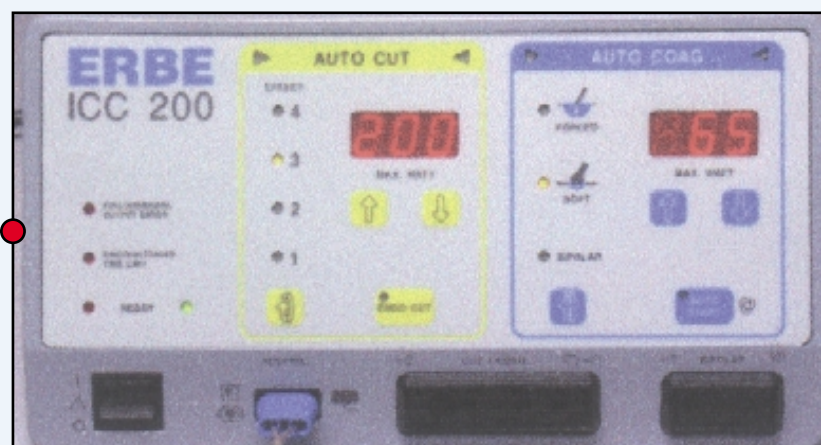

- Set AUTOCOAG to Soft
- Set Power: 60-70W
- Biopsy tool plugs in Mono port
- Use **Blue** Footswitch
- Patient plate on patient

BiCap™ Bipolar/Gold
Probe

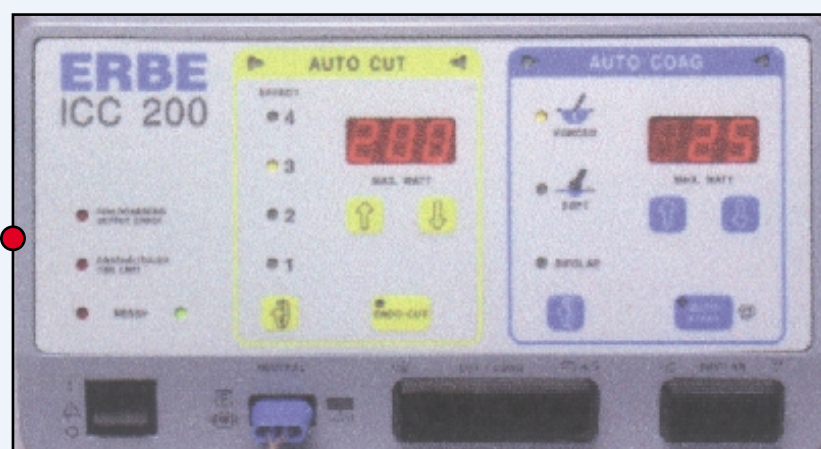

- Set AUTOCOAG to Bipolar
- Set Power: 15-30W
- Accessory into Bipolar Plug
- Use **Blue** Footswitch
- No patient plate